



# QuickAcuity Printable Near Vision Chart

Use at exactly 16 in / 40 cm. Line labels are 20/xx equivalents. Print at 100% or Actual Size.  
More printable tools: [quickacuity.app/printable-near-chart](http://quickacuity.app/printable-near-chart)

### Reduced Snellen near card

Hold the page flat at eye level with bright, even lighting.

20/200	E
20/100	F P
20/70	T O Z
20/50	L P E D
20/40	P E C F D
20/30	E D F O Z P
20/25	F E L O P Z D
20/20	D E F P O T E C
20/15	L E P P O P E T

### Print and verify

Print in portrait at 100% scale. Do not Fit to Page, Shrink, or use printer scaling.

Verify the 50 mm and 2 in rulers first. Reprint if either ruler is off.

Smooth paper, solid black ink, and even room lighting all improve reliability.


### Use

Hold the chart at exactly 16 in / 40 cm. Test one eye at a time unless you specifically want binocular reading screening.

Wear your usual reading correction if you want corrected near vision. Remove it if you want uncorrected near vision.

Screening only. New vision loss, pain, flashes, distortion, or double vision needs clinical evaluation.

Scale check: 50 mm (1.97 in)

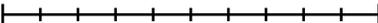

Scale check: 2 in

